



**INSTRUCTION MANUAL** **starLITE 3 HighPower**

For:	Your hearing system dispenser & Service Partner:
------	--

**You have chosen a hearing system from HANSATON**

We take pleasure in your choice and congratulate you on your decision, because with a HANSATON hearing system you own a high-quality product of the most modern technology.

For more than 45 years, we have been helping people with our devices to achieve better hearing and better speech understanding, hence a higher quality of life.

Behind our products stand many years of development and production experience, as a corporate philosophy centred on people and their needs.

If you read this brochure carefully and follow the advice on use and care, you will get much enjoyment from your new hearing system.



This product has passed every phase of the independently-certified HANSATON quality management system.


The CE symbol documents high quality standards and product reliability because it indicates that all European guidelines for medical products have been met.





**Index**

Technical Information for your hearing system	Page 2
Installation and change of the battery	Page 3
Information and safety instructions	Page 4
Handling of your hearing system	Page 5
Advice on care	Page 6
Advice on Use	Page 7
Special Equipment and Terms	Page 8
Remarks	Page 9

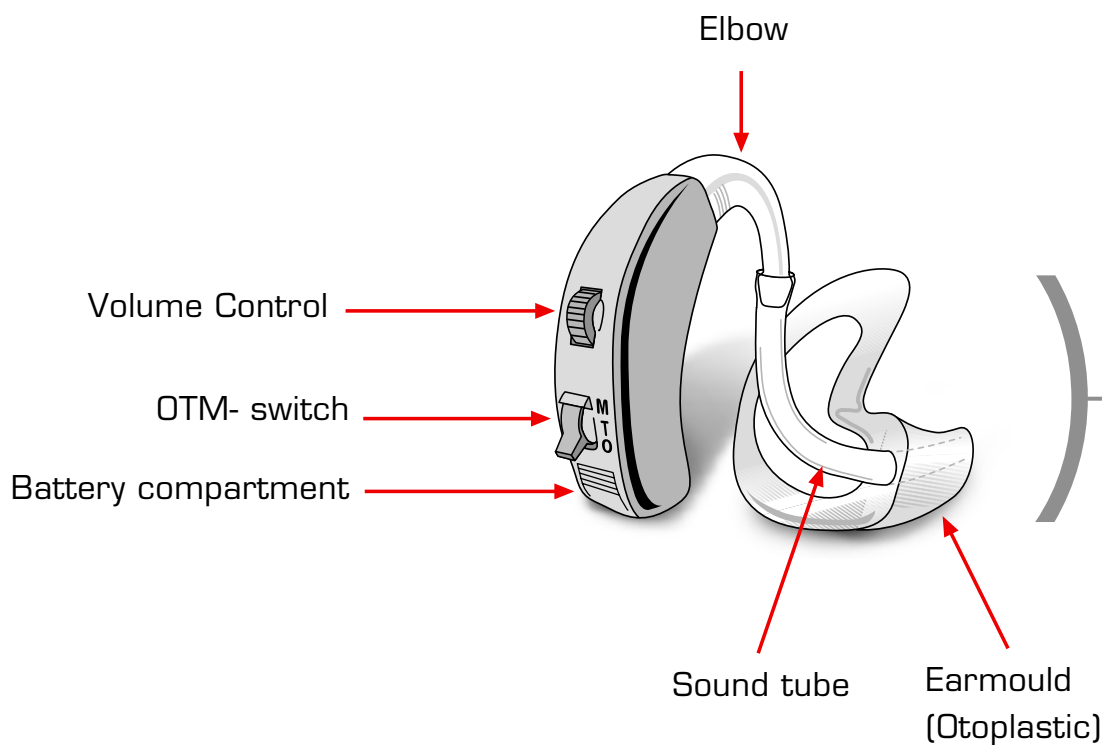




## Installation and Change of the Battery

Battery type: 675



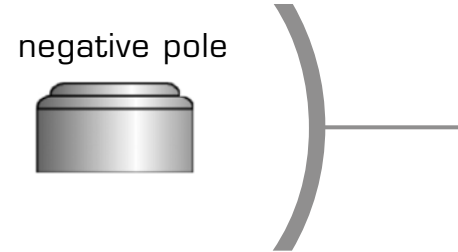



Open the battery-compartment and lay the battery in the open compartment so that the negative pole points into the compartment.




With a new battery, it may take a few minutes for the battery to develop its full output. Only then should you close the battery-compartment and switch on the instrument with the switch.



### **Advice on Batteries**

-  Never put batteries in your mouth ! For this reason, keep batteries out of reach of children.
-  If a battery is inadvertently swallowed, please call a doctor immediately.
-  Always use new batteries.
-  Remove the batteries if the hearing system is to remain unused for a long time.
-  Failing batteries should be replaced immediately.
-  Do **not** throw used batteries in the household waste or burn them, but return them to your hearing instrument specialist.

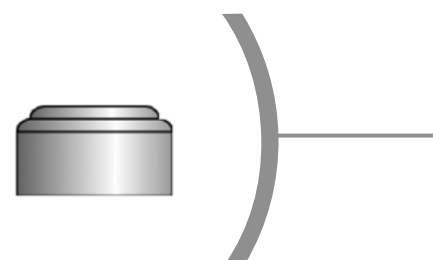

### **Safety instructions**

**Never attempt to recharge batteries that are not designed for that purpose (danger of explosion)!**





### Turning device on/off


Activate your hearing system by switching the OTM- switch to "M".

Position of the switch:

**M** = switches your hearing system on

**T** = allows inductive signal reception  
(e.g. while on the phone)

**O** = switches your hearing system off

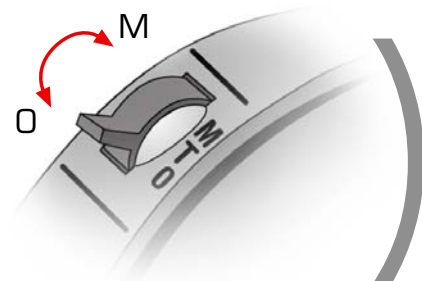


**M** = On (microphone)  
**T** = Telephone coil on  
**O** = Off

### Loudness setting

You can adjust the loudness of your hearing system infinitely variable from 1 to 4.

**1** = quietest position (wheel backward)


**4** = loudest position (wheel forward)

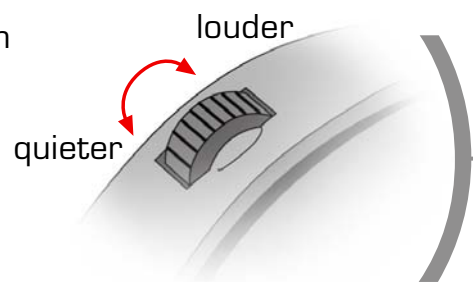



## Advice on care

- We recommend you use our HADEO care products for cleaning, products which were developed specifically for your hearing system. HADEO products can only be obtained from your hearing instrument specialist.
- Remove the sound tube from the HEARING SYSTEM and put the EARMOULD for at least 15 minutes into the HADEO cleaning liquid.
- The hearing system itself must **not** be placed in water or cleaning fluid.  
Clean your hearing system (preferably in the morning) with a soft cloth, keeping it dry.
- The hearing system must be kept dry.  
Please put it in a HADEO Dry Bag or Dry Cup overnight to extract any moisture from the inside of the instrument.

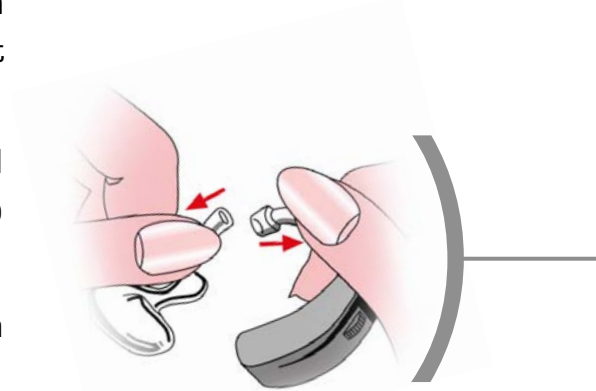

## Care Primer


Our HADEO care primer can be obtained from your hearing instrument specialist who will be glad to advise you on any questions you may have about caring for your system.






## Important Advice on Use

- Protect the device from wetness, dampness, heat and mechanical effects.
- Because of its sensitive components, the device should not be allowed to fall on the floor.
- Do not wear the device in the following situations:
  - > when showering/bathing
  - > when using hairspray
  - > in strong magnetic fields, high-frequency fields or under direct x-ray irradiation
- Please change the battery in the following instances:
  - > if volume becomes lower
  - > if sound quality deteriorates
  - > if the instrument switches off automatically  
(this will be preceded by a series of short-spaced alarms as a warning)
- HANSATON hearing instruments are so designed as to minimise the generation of electromagnetic fields and the influence of electromagnetic fields on their operation. When using mobile telephones, however, electromagnetic field effects cannot be ruled out.

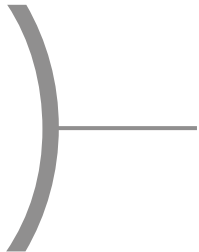

### **Safety instructions:**

Do not wear or use the device in areas where a danger of explosion exists.

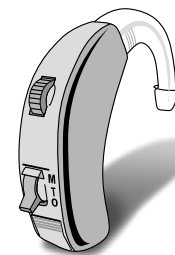




## Advice on Special Equipment


If you use a CROS / BICROS unit, it can cause a decline in the performance of the hearing instrument.

## Terms

- ➔ An **audio adapter** allows your hearing instrument to be connected to a signal source with the help of an audio cord. This must fulfill at least the safety requirements of IEC 60065 / DIN VDE 0860.

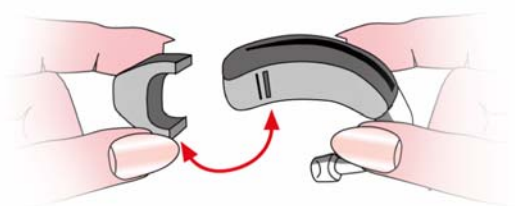

- ➔ Push the audio adapter onto the hearing system from underneath until it clearly engages.

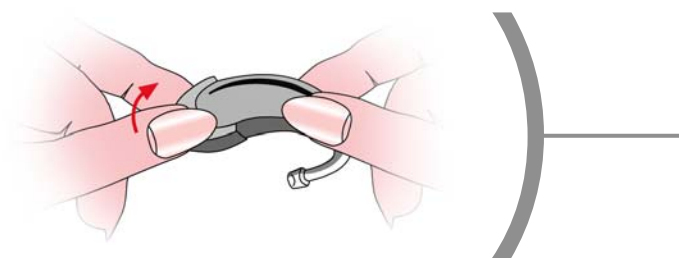

- ➔ Please lift off the Audio-Adapter with light, sidewise movements.

- ➔ Depending on the programming of your hearing system, you may be able to use the **telephone coil**. The telephone coil allows inductive signal reception (e.g. while on the phone). The reception of noise via the microphone is reduced or suppressed.

